

Instructions for Care

- **SEGUFIX®-Restraints** (white) are washable in hot wash cycles up to 95°C. Lower temperature and laundry nets will prolong product life. Avoid methods that might bend metal parts of the system (e.g. laundry line, dry pressing, rotary irons), eyelets may be damaged.
Not pre-washed, up to 8 % shrinkage. Store in a dry place.
Detergents: non-aggressive, no fabric softener, no bleach.
Dryer: delicate cycle at low temperature.
Disinfection: may be disinfected using chemothermal disinfection washing procedure (item 3.1.2 "catalogue of tested and approved disinfectants and disinfection procedures" published by the Robert-Koch-Institute). Overdosing can damage the product.
- **SEGUFIX®-Transport Belts** (coloured) are washable in warm wash cycles up to 30°C.
Avoid methods that might bend metal parts of the system (e.g. laundry line, dry pressing, rotary irons), eyelets may be damaged. Store in a dry place.
Detergents: non-aggressive, no fabric softener, no bleach.
Dryer: delicate cycle at low temperature.
- **SEGUFIX®-Locks and metal pins**, remove all Locks and metal pins before washing or drying. Laundering will damage Locks and possibly cause them to fail. Soiled Locks may be cleaned with a damp cloth. Never submerge Locks in any type of liquid.
- **Velcro® fastener**, fasten before washing and drying to prevent lint buildup. Occasionally use a stiff brush to remove lint from Velcro® fastener. Even with normal use, Velcro® material has the tendency to collect lint. This can reduce the grip strength. Before each use, test grip strength of the Velcro® fastener.
- **Durability** depends on the intensity of use and is prolonged by gentle laundry procedures (detergent/laundry net).
- **SEGUFIX®-Sheepskin products:**
Washing machine: Washable at warm wash cycles up to 30 - 40°C, no spin-cycle. Use only wool or delicate detergents.
Hand washing: Wash quickly in lukewarm water (30 - 40°C). Rinse thoroughly with lukewarm water and gently wring-out water. Use only wool or delicate detergents.
How to dry the sheepskin: Hang dry or tumble dry on delicate cycle at low temperature.
Caution: Avoid direct sunlight! Before the sheepskin is completely dry, pull to reshape it. If needed, brush with a sturdy brush.
Note: Do not leave sheepskin submerged for an extended period of time (do not soak).

Abbreviations for various versions

Version f - Fleece

- for sensitive skin
- skin-friendly padding
- pleasant and soft
- 100 % pure Cotton

Version r - reinforced

- eyelet straps reinforced for higher demands

Version e - extended bed belts

- for beds wider than 1 m / 40 inches, extended on each side by approx. 20 cm / 8 inches

Version v - Velcro® fastener / Buckle Lock

- positioning aid
- not a restraint*

*Note: However some jurisdictions might consider Velcro® products a restraint, should the patient not be able to remove them independently. Please check your applicable legal requirements.